KEEP MEDICAL

PRODUCT MANUAL

KM

— KN95 Face Masks

PRODUCT INFORMATION



Product Name: Respirator Cone Face Masks Category/Standard: EN149:2001 FFP2 NR D Shelf Life: 2 Years Production Date: See Marking

Material (outer to inner): 1st Ply – Outer Nonwovens 2nd Ply – Melt-Blown Cloth 3rd Ply – Hot Air Cotton 4th Ply – Melt-Blown Cloth 5th Ply – Inner Non-Woven Fabric

Cautions and maintenances

1. This Respirator Cone Face Mask is not allowed to wear/use and cannot protect you under circumstance of: temperature above 50°C, oxygen content less than 19.5%, expose to toxic gases (this is NOT a replacement of gas mask)

2. This product is only suitable for non-oil aerosols respiratory protection.

3. Children, pregnant women and elder people are NOT recommended to use.

4. Storage: suggest to keep indoor circumstance with humidity <80%, no corrosive gas and good ventilation.

5. Do NOT use if package is damaged.



PRODUCT INFORMATION

Way of Wearing

- 1. hold the ear loops with nose clip upwards.
- 2. wear the respirator at jaw.
- 3. fix ear loops and adjust to a comfortable way.
- 4. tap gently at center of nose clip and adjust it to well fit with your nose.
- 5. re-check and make sure your face is fully covered by respirator.



CERTIFICATE OF COMPLIANCE

CERTIFICATE OF COMPLIANCE

This product has been inspected and tested and proved to be qualified for delivery in conformity with standard.


Product Name: KN95 Face Masks Material: spun-bond, hot air-through and melt-bond non-woven Color: solid Product standard: GB2626-2006 Production Date: refer to package

Shelf Life: 2 years Inspector:

s QC PASS 01

MADE IN CHINA

WAY OF PACKING

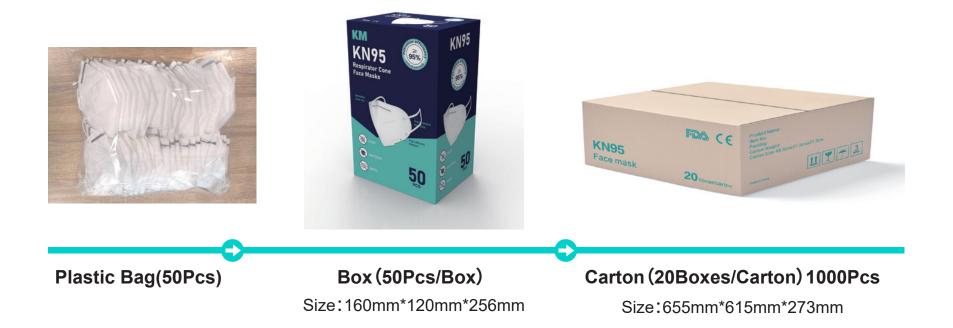

BUSINESS INFORMATION



KM

MOQ: 100K Payment Terms: T/T before shipment

Banking Info: Account Name: DONGGUAN KEEP MEDICAL TECHNOLOGY CO., LTD. Account Number:44300001040024668 Bank Name:Agricultural Bank of China, Qishi, Dongguan Branch.

Transportation and Express Info: Pick-up in factory is recommended for massive orders. SF express with freight collect is the default option unless other options agreed prior.

Company Info: Company Name: DONGGUAN KEEP MEDICAL TECHNOLOGY CO., LTD. Registration Number: 91441900MA54BFMHXA Address: No.31 Mingyingyuan Road,Qinghu,Qishi Town, Dongguan City,Guangdong,China



TEST REPORT (English)

	ion Centre (Zhej	iang)	
	Test Report		
DDRESS: No.300 XIASHA ROAD ANGZHOUCHINA el: (0571) 85122669 ax: (0571) 85125121 osteode: 310018		NUMBER:W2 DATE:2020-04-03	02003520E
PELICAN CONGGUAN KEEP MEDIC AMPLE DESCRIPTION: KN95 Face Mas DDRESS No. 31, Qinghu Private Garden	k		Province
ITEM	STANDARD (KN95)	RESULT	RATING
I. FILTER EFFICIENCY(SALT MEDIUN GB 2626-2006)(%)	4)		anti ili
	≥95.0	99.2	PASS
2. RESPIRATORY RESISTANCE (GB 2626-2006)(Pa)			
-INSPIRATORY RESISTANCE	≤350	142	PASS
EXPIRATORY RESISTANCE	≤250	131	PASS
PICTURE -		A C	
PICTURE	I		
PICTURE enurks: The pretreatment conditions were	: 38°C, 85% humidity for	24h.	
	: 38°C, 85% humidity for	24h.	
emarks: The pretreatment conditions were	: 38°C, 85% humidity for		
emarks: The pretreatment conditions were \overrightarrow{rr} <u><u>t</u> </u>	17777 · ···		
emarks: The pretreatment conditions were \overrightarrow{rr} <u><u>t</u> </u>	17777 · ···		

1

TEST REPORT (Chinese)



National Textiles and Garment Quality Supervision Inspection Center(Zhejiang)



TEST REPORT (Chinese)



TEST REPORT (Chinese)





$\textbf{TEST REPORT} \hspace{0.1 cm} (\textbf{Chinese})$



$\textbf{TEST REPORT} \hspace{0.1 cm} (\textbf{Chinese})$

声明	
一、本机构保证检测的公正性、独立性和诚实性,对检测结果负责,对委托方所 二、本报告无签发人员签字或未加盖本机构红色检验检测专用意无效、 三、报告流改无效、契印件未重新加益本机构红色检验检测专用意无效、 四、本报告的检测数据和检测反对达检律出负责。	提供的检测样品保密和保护所有权。
五、本报告各页均为报告不可分割之部分,使用者单独抽出某些页导致误解或用于 构不负相应的法律责任。	
六、委托方若对本报告有异议,应及时向本机构提出。政府行政管理部门下达的非 议时,应按政府行政管理部门文件规定及国家相关法律、法规规定进行。	首令任忙旁,做位力对位短跆来有并
DECLARATION	
 Our organization guarantees impariality, independence and honesty of inspection, feasing the samples supplied by the entrusting party confidential and at the the samples supplied. The test report will be deemed invalid without signatures of the inspector/reviewer special inspector is stamp of our organization. 	same time protecting the ownership of and authorized personnel, and the red
3. The test report will be invalid if it is altered. Copies of the report are invalid without organization. 4. The test results shown in this report is only valid for the tested sample. 5. All the pages of the report are integral parts of the report. Our organization misunderstanding or other results cusued by using separate page(s) of the report. 6. If there is any dissent of the report, the entrusting party shall notify our organization given by governmental administration departments, and dissent about the sample bein should be dealt with in accordance with national regulations.	on will not be responsible for any timely. For the mandatory inspection
浙江省轻工业品质量检验研究院及附设的检	验中心
The Affiliated Inspection Centers	5, 80
浙江省轻工业品质量检验研究院 Zhejiang Light Industrial ProductsInspectionandResearchInstitute	联系电话: 0571-85122669 Telephone: 0571-85122669
国家防闭服装产品质量监督检验中心(浙江) National Textiles and Garment Quality Supervision Inspection Center(Zhejiang) 地址:浙江省杭州市江干区下於第 300 夕ら 6 号楼 Address: Building No.6, 300 XiaSha Road, Hangzhou, Zhejiang	E-Mail: 1137907994@qq.com
国家家具产品质量监督检验中心(浙江) National Center for Quality Supervision Inspection of Furniture(Zhejiang) 浙江省室内安全及家具产品质量检验中心	联系电话: 0571-89009556 Telephone: 0571-89009556
Zhejiang Center of Quality Test for Indoor Safety and Fumiture Products 地址:浙江省杭州市余杭区良渚街道经一路1号良渚大学科技园4号楼	E-Mail: 2047699564@qq.com
Address: Building 4 LiangZhu University Science and Technology Park, No.1, JingYi Rd, Yuhang District, Hangzhou, Zhejiang	
浙江省轻工及五金产品质量检验中心	联系电话: 0571-89001107
Thejiang Center of Quality Test for Light Industry and Hardware Products 浙江省体育用品质量检验中心	Telephone: 0571-89001107
Zhejiang Center of Quality Test for Sports Products 地址:浙江省杭州市西湖区天目山路 222 号 3 号楼	E-Mail: wujtest@126.com
浙江省杭州市金杭区良渚街道经一路1号良渚大学科技园3号楼	
浙江省杭州市余杭区良渚街道经一路1号良渚大学科技园3号楼 Address: No. 222 Tianmashan Rd., Hangzhou, Zhejiang Building 3 LiangZhu University Science and Technology Park, No.1, JingYi Rd., Hangzhou, Zhejiang	
Address: No. 222 Tianmushan Rd. Hangzhou, Zhejiang Building 3 LiangZhu University Science and Technology Park, No.1, JingYi Rd., Hangzhou, Zhejiang 国家領具产品景監督経験中心(浙江) National Center For Quality Supervision Inspection of Lock/Zhejiang)	联系电话: 0571-85027738 Telephone: 0571-85027738
Addresss, No. 222 Tianmushan Rd., Hangzhou, Zhejiang Buiking S LiangZhu Livirexrity Science and Technology Park, No.1, JingYi Rd., Hangzhou, Zhejiang 国家頓具产品质量盒督检验中心(浙江) National Center for Quality Supervision Inspection of Lock(Zhejiang) 浙江省現中活動量检验中心(浙江) Zhejiang Center of Quality Test for Lock Products 地上。浙江省低州市增高及 24 년	
Address: No. 222 Tianmushan Rd., Hangzhou, Zhejiang Building 3 LiangZhu University Science and Technology Park, No.1, JingYi Rd., Hangzhou, Zhejiang 国家領具产品质量監督經會中心。(浙江) National Center for Quality Supervision Inspection of Lock(Zhejiang) 浙江?會現其产品质量監督經中心(3世江) Zhejiang Center of Quality Test for Lock Products	Telephone: 0571-85027738



EXPORT & IMPORT REGISTRATION

经营者中文名称	东莞市凯医科技有限公司			
经营者英文名称	DONGGUAN KEEP MEDICAL TECHNOLOGY CO., LTD			
组织机构代码	10000000000000000000000000000000000000	经营者类型 (由备案登记机)		
住所	广东省东莞市企石	广东省东莞市企石镇清湖民营园路31号		
经营场所 (中文)	广东省东莞市全石镇清湖民营园路31号			
经营场所(英文)	No.31, Qinghu private garden road, Qishi Town, Dongguan City. Guangdong Province			
联系电话	13751342815	联系传真		
邮政编码	523000	电子邮箱	413626708@qq. com	
工商登记注册白期	2020-2-20	工商登记注册	5. A <u>NA BOOM</u>	
大法办理工商登记的企业	业还须填写以下内容	ANT SA	U. AXE SAL	
企业法定代表人姓名	朱凯敏	有效证件号	33041919731126205	
注册资金	在佰万元	SEE C.	(折美元)	
大法办理工商登记的外国	国 (地区) 企业或个体	工商户 (独资经)	雪者)还须填写以下内容	
企业法定代表人/ 个体工商负责人姓名	Same and	有效证件号	Statistics and the second s	
企业资产/个人财产	All and	aller and a second	(折美元)	
备注				
	均条款、并由企业法定	代表人或个体工	前负责人第字5 曹雪 秦家登记机关***	

CERTIFICATE OF APPROVAL FOR MEDICAL DEVICES

20 53.	第二类医疗器械经营备案凭证 _{各类编号1} 考东食药监核经营备20200397号
企业名称	东莞市凯医科技有限公司
法定代表人	朱凯敏
企业负责人	杨伟
经营方式	批零兼書
住所	广东省东莞市企石镇清湖民营圆路31号
经营场所	广东省东莞市企石镇清湖民营园路31号
库房地址	广东省东莞市企石镇清湖民营园路31号
经营范围	2002年分类目录:6820, 6864 2017年分类目录:14
	备案部门(公章) 按加监查 编》 备案日期: 2020年3月 2030日

KEEP MEDICAL